



Thank you for choosing RSM Diagnostics Lab.
Please contact the lab at (302) 592-4106 from
8:30am-4:30pm EST, Monday-Friday, with any questions.

**Please read and follow these instructions carefully BEFORE
collecting stool sample to ensure accurate results!**

STEP 1: Write patient last name, date of birth, and collection date on the specimen bottle.
Complete ALL of the highlighted areas on the lab order or Test Requisition Form (TRF) including:

- All patient information
- The date and time of sample collection
- If the patient is a minor/under 18 the name of the parent who collected the sample.
- Payment information (if indicated).

Missing or illegible information on the TRF can result in a hold on releasing result, so please fill out the form carefully and completely, and return with the sample!

STEP 2: Wash/dry hands, put on gloves, and place stool sample collection sheet on toilet by removing tape to expose adhesives on each side of the sheet and sticking it to each side of the toilet. (See reverse side for detailed instructions.)

STEP 3: After passing stool in the collection sheet, unscrew bottle and collect stool using the spoon attached to the cap of the bottle from 3 different places in the feces.

STEP 4: Mix the stool collected on the spoon into the liquid/gel medium at the bottom of the specimen bottle. **Add enough stool to fill the tube up to, but not above, the red line.** Replace cap tightly and shake vigorously for 30 seconds.

STEP 5: Place the specimen bottle back into the specimen biohazard bag. Place the lab order or TRF in the document holder in the back of the specimen biohazard bag (*this form must be completed and returned to process the sample*). Place the specimen biohazard bag back into the white, bubble envelope that was shipped to you. Place the envelope into the pre-labeled UPS, FedEx or USPS shipping Clinical/Laboratory Pak and seal.

STEP 6: Your test results will be delivered to your medical provider **within 2-3 weeks after the sample(s) are received at RSM Diagnostics Lab.**


Shipping Instruction: Please take sealed Clinical/Laboratory Pak to the nearest corresponding shipping location for drop off **WITHIN 24 HOURS of collection.** (Tracking can be found on the outside of the return bag.) Shipping is best done on Monday thru Wednesday to insure the sample arrives at RSM before the weekend.

Note: Do NOT ship Thursday - Sunday. If specimen is collected on these days, REFRIGERATE (do not freeze) specimen and ship the following business day/Monday.

Raku-Ryu Cup Wide

Instruction manual


Step1. Preparation

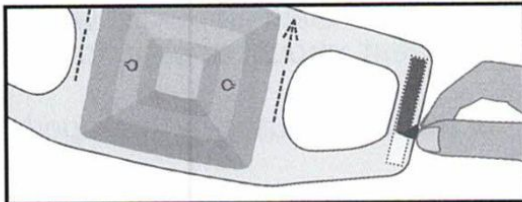


Peel the paper lining from the back side of the double-sided tape.

Step2. Setting


Place the sampling sheet onto the toilet seat or the bowl.
※please use it for a dry toilet seat.

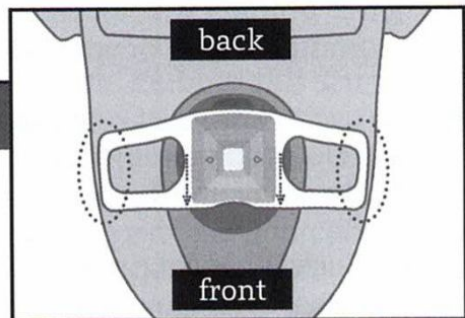


Setting height

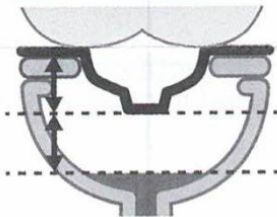

Setting image



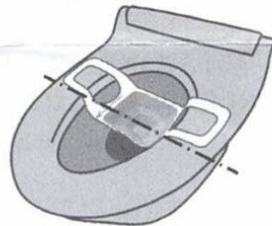


Setting direction

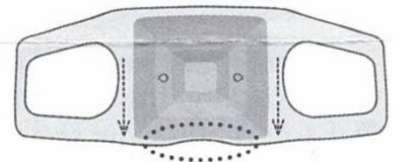

The notched area should face the front of the toilet

Step3. Flush after Stool Sampling

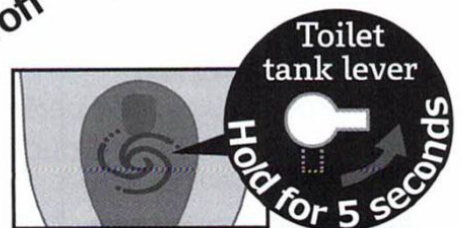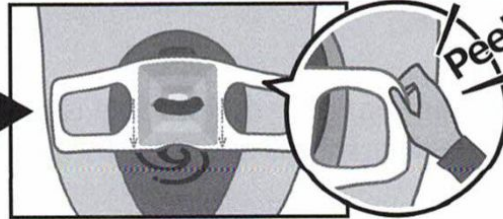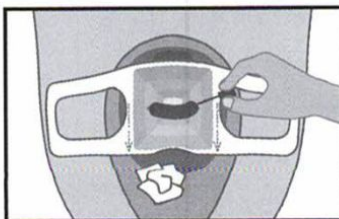

After toilet use, test the stool sample.
At the first, flush the toilet paper accumulated in the toilet bowl using sufficient amount of water to flush the faces, then flush this product that feces are caught using another sufficient amount of water to flush the feces.

Scan QR Code below to watch video instructions.

When flushing, in any case.

Please tilt the toilet tank lever enough to flush the stool.
Please hold for 5 seconds as it is.

